

Skinnia Wounds



Apósito cicatrizante en gel



Indicación

- Protección de heridas quirúrgicas
- Cortes
- Excoriaciones
- Rozaduras en caballos

Modo de empleo

Aplicar una fina capa de gel suficiente para cubrir completamente la zona afectada tres veces al día. Esperar aproximadamente un minuto a que el producto se seque para que se forme el film.

Composición

Eanol, Polímeros filmógenos.

Especies de destino

Perro, gato y caballo.

Modo de acción

Al ser aplicado, el alcohol ejerce una acción desinfectante inmediata, evaporándose en cuestión de segundos y favoreciendo así la formación del film protector sobre la piel. Además, gracias a la excelente combinación de excipientes, favorece el proceso de cicatrización acelerando así la curación de la piel dañada.

Formato

Tubo de plástico con cánula de 30g.

Precauciones

Mantener fuera del alcance y la vista de los niños. Evitar contacto con ojos y mucosas.

Principales ventajas



Fácil aplicación
Zonas de difícil acceso



Película transparente
Vigilancia directa



No se elimina de la
zona de aplicación



Aislamiento de la herida
Alta permanencia




Barrera transpirable
Evita la maceración

CASO DE USO

Nombre: BRUNO | **Raza:** GOLDEN RETRIEVER | **Edad:** 9 AÑOS | **Estado:** HERIDA POR MORDISCO



Día 0

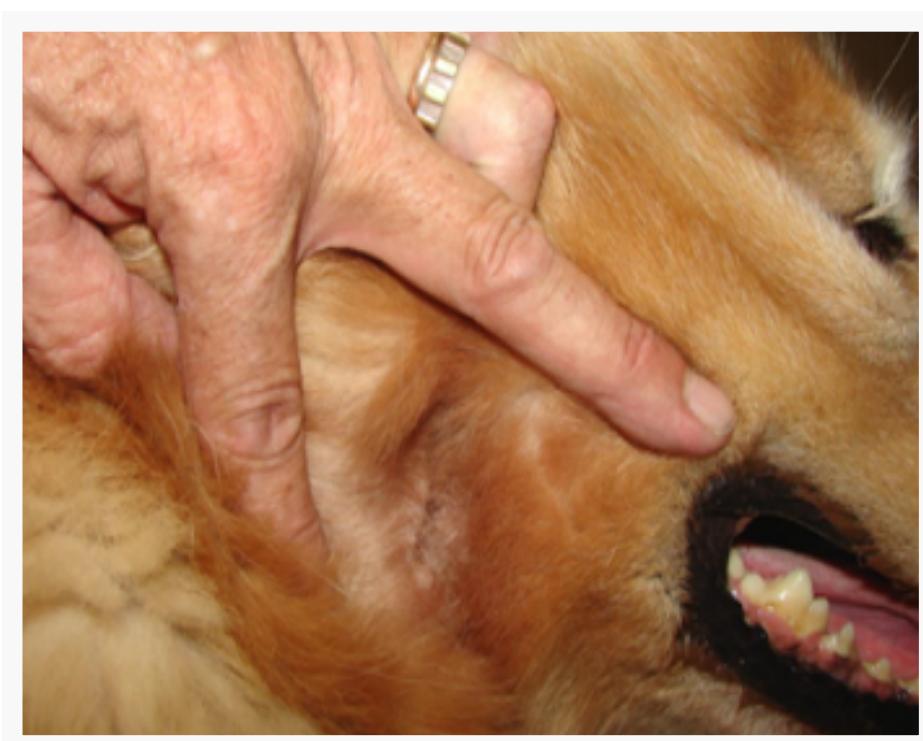

Día 7

Skinnia Wounds



Healing film-forming gel



Indication

- Surgical wound protection
- Cuts
- Excoriations
- Chafing on horses

How to use

Apply a thin layer of gel sufficient to completely cover the affected area three times a day. Wait approximately one minute for the product to dry to form a film.

Composition

Ethanol, Film-forming polymers.

Target species

Perro, gato y caballo.

Mode of action

When applied, the alcohol has an immediate disinfectant action, evaporating in a matter of seconds and thus promoting the formation of a protective film on the skin. In addition, thanks to the excellent combination of excipients, it promotes the healing process, thus accelerating the healing of damaged skin.

Format

30g Plastic tube with cannula.

Cautions

Keep out of the reach and sight of children. Avoid contact with eyes and mucous membranes.

Advantages



Easy application
Areas of difficult access



Transparent film
Direct surveillance



It is not removed from
the area of application



Wound isolation
High permanence



Breathable barrier
Prevents maceration

STUDY CASE

Name: BRUNO | **Breed:** GOLDEN RETRIEVER | **Age:** 9 YEARS | **Condition:** BITE WOUND



Day 0



Day 7